

**evrclear**<sup>™</sup>  
mono crystalline bracket  
De-Bonding Pliers



## De-Bonding Plier Instructions

- 1) Place the plier on the bracket both side like the picture.
- 2) Gently squeeze the plier's handle.
- 3) Move left side and right side with the squeezed bracket.



1300 NE Alpha Drive  
McMinnville, OR 97128  
www.orthoClassic.com

Toll Free: 866.752.0065  
International: 503.472.8320  
Fax: 866.752.0066

Star Dentech Korea, Corp.  
Tel: 82 2107 7111 / Fax: 82 2107 7112